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AMALON

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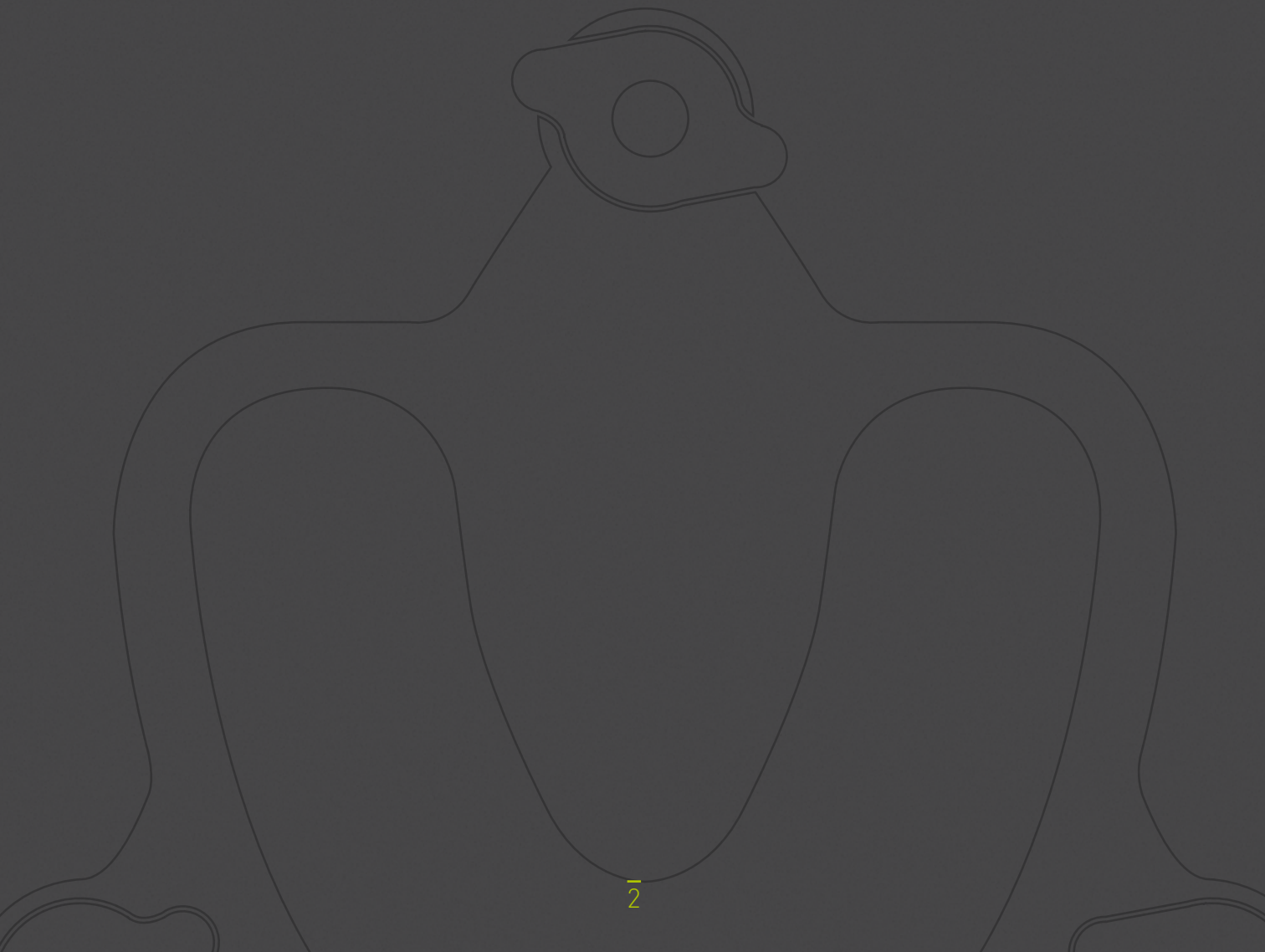
ESSENTIAL DAILY WORK

“

 Fast Protec has an essential nature, since it synthesizes long and expensive traditional processing in a fast, clean and easy way of working. We invest in technology and regular research for constant innovation as aesthetics, ergonomics and functionality play decisive roles in reaching simplicity. As our approach always starts with dental technicians needs, we created a complete system, made up of flasks, articulators, verticulators, resins and silicones to perform all dental processing. This is Fast Protec: precise protocols that will guide you step by step to make your workflow faster and more organized.

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Flasks Muffeln Muflas

ALU BIG

 Alu Big is a flask made of aluminium with an ergonomic design. By using silicone templates, it is possible to pour, inject or press the resin to obtain in a fast and precise way the most part of dental processing as removable dentures, relines, removable partial dentures, crown and bridges, surgical templates and implant dentures.

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IMPLANT BOX

 Implant Box is a flask with a minimal and compact design that allows you to cast the model in plaster directly on the base. Thanks to the integrated split cast it is possible to perfectly fit the base on Fast Protec Articulator, Verarte and All-In. It is provided with two tips: the metallic one is useful to perform crown and bridges and implant dentures whereas the clear one is perfect to work with light-curing composites.

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BIG AND SMALL GLASS

 Big and Small Glass are transparent flasks with a POM base and two clear lids held by secure locking screws to ensure optimal results and no increases in the vertical dimension. The sizes of these flasks make them compatible with any polymerization chamber available on market and the double lids have been designed for the duplication of crown and bridge wax-up

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SMALL GLASS XL

 Its compact shape, makes the Small Glass XL the ideal flask to realize works from 1 to 6 elements. Thanks to its additional lid, this flask allows technicians to perform implant dentures, joining the advantages of the Small Glass with those of the Light Glass, reducing any waste of Silicone and working on teeth and gingiva.

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LIGHT GLASS

 Wider than traditional transparent flasks and more stable with its three screws, Light Glass allows dental technicians to perform full implant dentures working simultaneously on composite teeth and gingiva thanks to the provided three lids and the removable three pins. Its completely clear structure guarantees a fast and total light-curing of the elements obtaining perfect and precise details.

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ARTICULATOR

 The unique articulator of this system is provided with magnetic condyles fixed at the standard 30 degrees to perfectly reproduce all mouth movements. Diagnosis and treatment planning are simplified thanks to the posterior incisal guide pin that gives total visibility at 360°. Functional and easy to use, Articulator enable to set the model directly in articulation through the use of silicone instead of plaster.

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VERARTE

 Verticulator and Articulator combined in one unit in a revolutionary design. The posterior rod pin guarantees a perfect and complete visibility of the prosthesis and simply by changing the upper support of the Articulator, it is possible to transform it in a verticulator to perform ceramic moulding or plan implant dentures.

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ALL IN

 All-In is a combination of verticator and flask composed by one base with three rod pins and two interchangeable lids. The equilateral base enable the rotation of the model and the template at 360° allowing the dental technician to have the maximum visibility of his work. Essential item for the pre-visualization of the case and the casework planning as a verticator, it could be used for the injection or pouring technique as a flask.

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Implantology Implantologie Implatologia

EASY BAR

 Easy Bar is a kit of preformed bars that facilitate the realization of a stable, precise and reinforced structure for all kind of fixed dental prosthetics in just 6 hours. Thanks to their different sizes (from 6 to 30 mm), these bars could be adapted by bending to any position of the implants and are compatible to every implant abutment on the market. Available in **steel** or **titanium (NEW)**.

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JIG PLATFORM

 Jig Platform is ideal for positioning and aligning drill holes for abutment holes in dentures. It is provided with a secure magnetic attachment on the base and an integrated split cast.

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VICTORIA

 Compact, easy and functional polymerizing machine. It is possible to cook cold and heat-curing resins with water input and to harden silicone templates with manual air loading through button up to 6 bar pressure. Victoria is provided with heat strips that heat water gradually until a maximum temperature of 120 degrees and an automatic counter that starts decreasing time once the pre-set temperature is reached maintaining it constant.

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NEW LIGHT BOX

 The New Light Box is a compact and modern polymerization unit provided with last generation multi-die LEDs for light-curing composite resins. A low consumption machine that works on a vacuum pump to avoid the dispersion layer, the New Light Box is perfect to light-cure the Fast Protec wide range of composites at a wave frequency between 360 and 520 nm.

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Accesorios

DOUBLE BASE

 Two base formers made of aluminium with rubber cuffs useful to cast the model in plaster. Thanks to the split cast designed on the base, the model becomes compatible with any product of the Fast Protec System fitting in a precise and clean way.

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ALU BIG CLEAR TOP

 This clear top transforms Alu Big in a transparent flask to work with light-curing composites.

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TEETH CLEAN

 Basket with 14 individual compartments for steam cleaning teeth securing each denture tooth position.

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MODEL PLATE

 Model Plate is a positioning device for setting edentulous models on the articulator.

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CLEAN TEMP

 Split cast base for temporaries crowns and bridges. Minimum waste of material thanks to its three undercuts.

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BASE FOR PLASTER

 Base for adapting any articulator on the market to the Fast Protec System.

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UNIVERSAL CLEAN

 Universal Clean is a little base that helps fitting the plaster model on the Articulator through the use of Universal Blue Silicone.

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UNIVERSAL BASE SPLIT CAST

 Little split cast base to adapt other bases available on market to the Fast Protec system.

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Silicones Silikonen Siliconas

UNIVERSAL BLUE SILICONE

 Kneadable silicone for the first templates. It guarantees an extremely precise detail reproduction without distortion even at high temperatures and an excellent long-term dimensional stability.

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Content: 1kg A + 1 kg B; 5 kg A + 5 kg B

PUTTY SILICONE

 With ideal handling characteristics and times, Putty Silicone is an economic rigid bulk filler silicone.

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50
SHORE

TRASPA SILICONE

 Crystal-clear silicone perfect for processing with light-curing composites.

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Content: 1 kg A + 1 kg B

REGULAR PRECISION

 Auto-curing and precise silicone useful to reproduce temporaries caps or for temporaries restoration.

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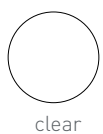
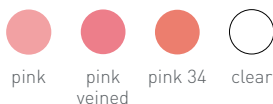
Acrylics Kunststoffe Resinas

ACRYLIC FOR DENTURES MICRO PEARL

 Cold-curing resin for the injection, pouring or pressing techniques. Different colours available but the same stability and no shrinkage to meet all your needs: full and partial dentures, repairs and relining could be performed in an easy and fast way. Top aesthetic prosthesis with less residual monomer.

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Content: powder 1kg; liquid 500ml

TEMP POWDER AND LIQUID

 High-tech acrylic for long lasting crowns and bridges . Temp Powder is a cold-curing resin with a high precision and abrasion resistance that avoids plaque deposits.

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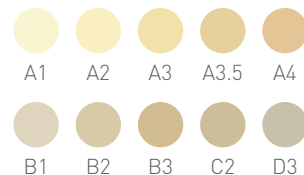
Content: dentine 100 gr; incisal 100 gr; liquid 100 ml

Dentine:

Incisal:

GENGICOLORS KIT

 Specifically studied for Fast Protec acrylic micro pearled resins, Gengicolors is a kit composed by two pre-mixed light and dark cold-curing resins and five stain colours for the characterization of marginal and attached gingivas. Optimal to give dentures a natural and individualized look.

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Content: light pink powder 100 gr; dark pink powder 100 gr; liquid 100 ml; stains 10 gr

light pink dark pink

blue red white brown yellow

red

UNIPATT RESIN

 Self-curing castable resin for telescopes, super- and supra-structures using the brush technique.

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MASTER POLYURETHANE

 Polyurethane resin ideal for fabricating precise and resistant models as well as crowns and bridges. Flowable and easy to trim, Master Polyurethane has a dimensional stability that guarantees precise details of all works with a shrinkage percentage close to zero.

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Isolating Agents Isolierung Aislantes

FAST ALGINATE

 Separating agent, based on alginate, useful to isolate the plaster model before injecting or pouring the resin.

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Content: 500 ml

SILICONE SPRAY

 Separating spray excellent to insulate silicone templates.

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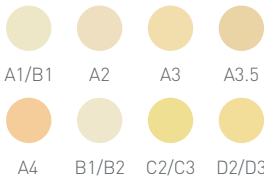
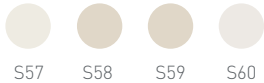
Effects:

COMP FLOW

 Light-curing flowable composite resin easy to inject, finish and polish perfect for inlays, onlays, implant-supported super structures, laminate veneers and long term temporary restorations. Its fluorescence and opalescence guarantee individual and natural aesthetic results respecting functionality and biocompatibility. Using Fast Protec transparent flasks it is possible to realize few elements or a complete arch in a fast and absolutely precise way.

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GENGIVA FLOW

 Light-curing gingiva composite ideal for gingiva restorations and reproductions, implant suprastructures, crowns and bridges. Thanks to Fast Protec wide range of colours, it is possible to characterize full or partial dentures with incredible final aesthetic results.

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Content: 1 syringe 3g

COMP METAL PRIMER

 Metal Primer is a perfect bonder between acrylics or composites and all types of metal.

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Content: 5ml

COMP METAL CONNECTOR

 Light-curing composite that improves the chemical bonding between primer and opaquer with a simple and secure adhesion.

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Content: 1 syringe 3g

BOND OPAQUER PINK DARK

 Light-curing paste opaquer useful to cover model casts with an integral adhesive component and perfect to cover Easy Bar frameworks before the injection of the resin.

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Content: 1 syringe 3g

COMP BONDING

 Conceived for the Fast Protec composite line, this bonding is a light-curing material for extensions and repairs of crown and bridge composites.

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Content: 7ml

COMP LACQUER AND EVER LAQUER

 Light-curing sealing lacquers for applying to the surfaces of denture acrylic, special tray material and composite crowns and bridges. Comp Lacquer is useful for temporary works whereas Ever Lacquer is a nifty product for permanent dentures.

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Content: Comp Lacquer 10ml; Ever Lacquer 5 ml

COMP COVER

 Masking gel that helps avoiding the dispersion layer when light-curing crown and bridge composites.

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Content: 4g

INTENSIVE COMP KIT

 Fast Protec offers a wide range of light-curing colours to characterize acrylic and composite teeth. Intensive Comp stains are easy to use because creamy and could be mixed to guarantee a natural and individual look according to the patient needs. The kit is provided with eight stain colours, a cleaner and a primer.

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
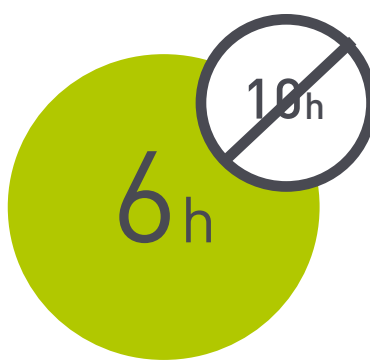
-  TRASFORMER® FAST PROTEC
-  TRADITIONAL METHODS

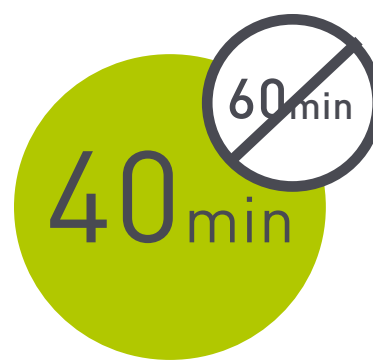-  Full Denture
-  Totaleprothese
-  Prótesis Móvil

-  Immediate Loading
-  Sofortbelastung
-  Carga Inmediata

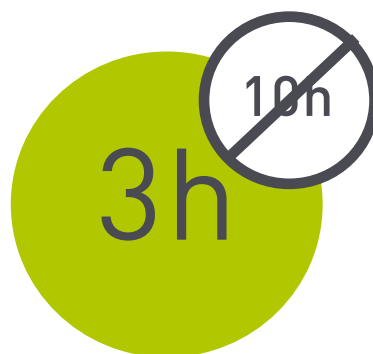
-  Temporary Crowns and Bridges
-  Provisorien
-  Provisionales

-  Ceramic Moulding
-  Keramikgießen
-  Estampe de la ceramica

-  Orthodontic Splint
-  Beißschiene
-  Férula

-  Implant Denture with Composite
-  Toronto im Komposit
-  Toronto en composite

Full Denture Totaleprothese Prótesis Móvil

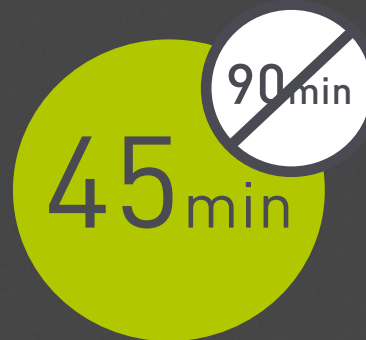
Performed by: Dt. Vincenzo De Feo

PRODUCTS USED:

- Alu Big
- Double Base
- Articulator
- Teeth clean

MATERIALS USED:

- Acrylic for Dentures Micropearl
- Universal Blue Silicone 70 Shore A+B
- Putty Silicone 70 Shore A+B
- Fast Alginate
- Gengicolors Kit

1. Wax-up
2. Occlusion check on the Articulator
3. First silicone template on the Alu Big
4. Second template
5. Teeth degreasing
6. Repositioning of the teeth on the silicone template
7. Isolation of the model
8. Closing of the flask
9. Resin pouring
10. Opening of the flask
11. Polishing
12. Final occlusion check

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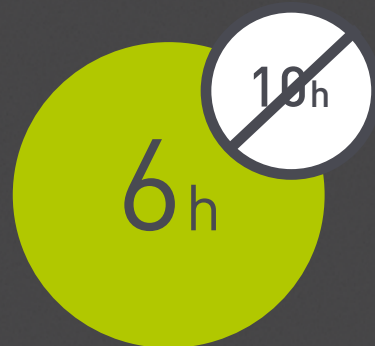
Immediate Loading processing shown in this page has been performed by Dt. Umberto Scilipoti

1. Implant denture model
2. Control of the bar position (previously fixed in mouth)
3. Model set on Articulator
4. Easy Bar structure welded
5. Easy Bar structure cemented on the abutments
6. Wax-up
7. Silicone templates
8. Opacified structure
9. Teeth insertion
10. Injection of the resin
11. Opening of the flask
12. Final result after polishing

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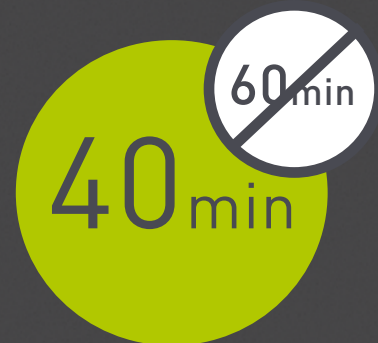
Performed by: Dt. Francesco Raffo

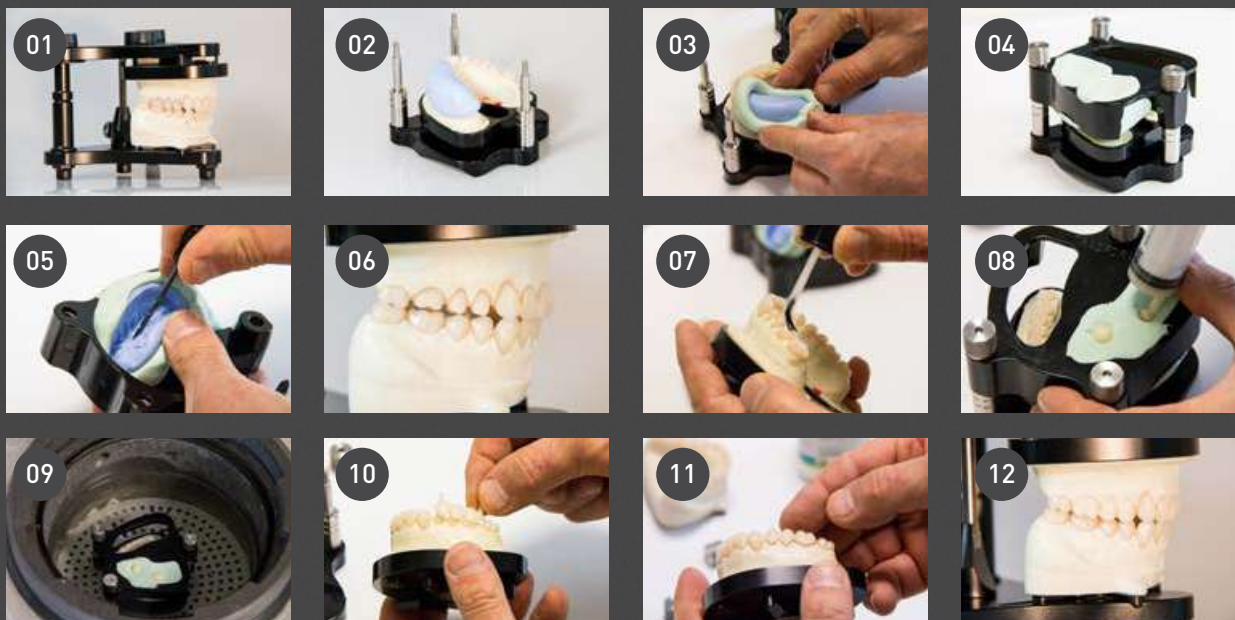
PRODUCTS USED:

- Alu Big
- Double Base
- Base for Temp
- Articulator

MATERIALS USED:

- Temp Powder
- Temp Liquid
- Universal Blue Silicone 70 shore A+B
- Putty Silicone 70 shore A+B

1. Occlusion check on the Articulator
2. First template on Alu Big flask
3. Second template
4. Closing of the flask
5. Injection holes
6. Occlusion check on the Articulator
7. Isolation of the model
8. Injection of the resin
9. Polymerization of the resin
10. Flask opening
11. Polishing
12. Final occlusion check

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

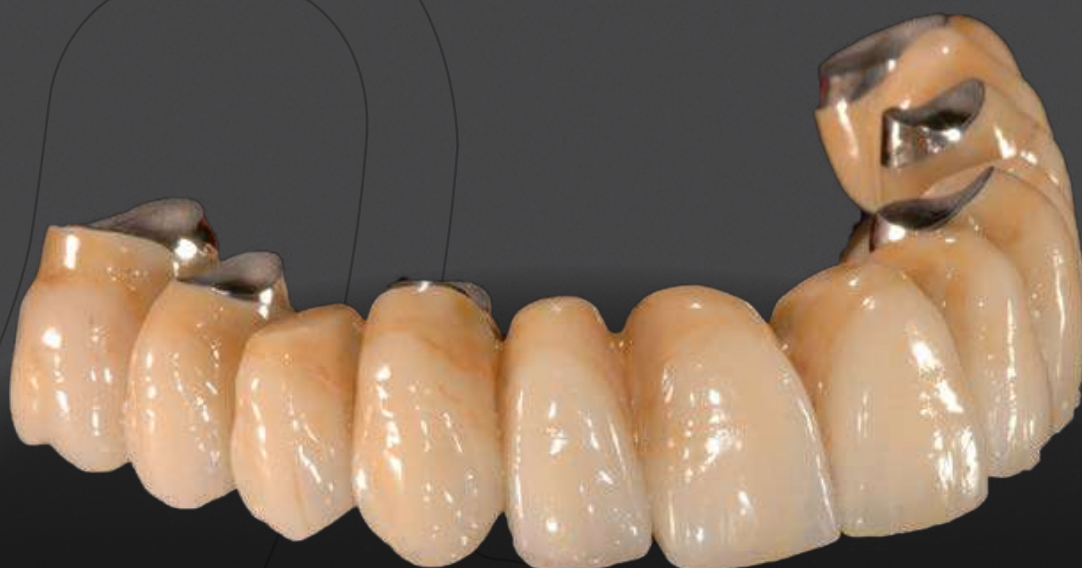
MATERIALS USED:

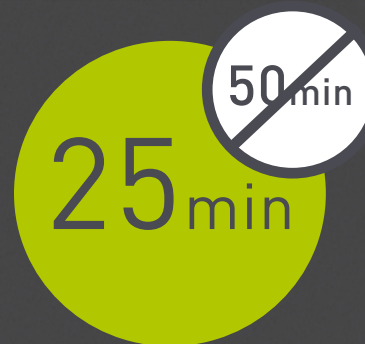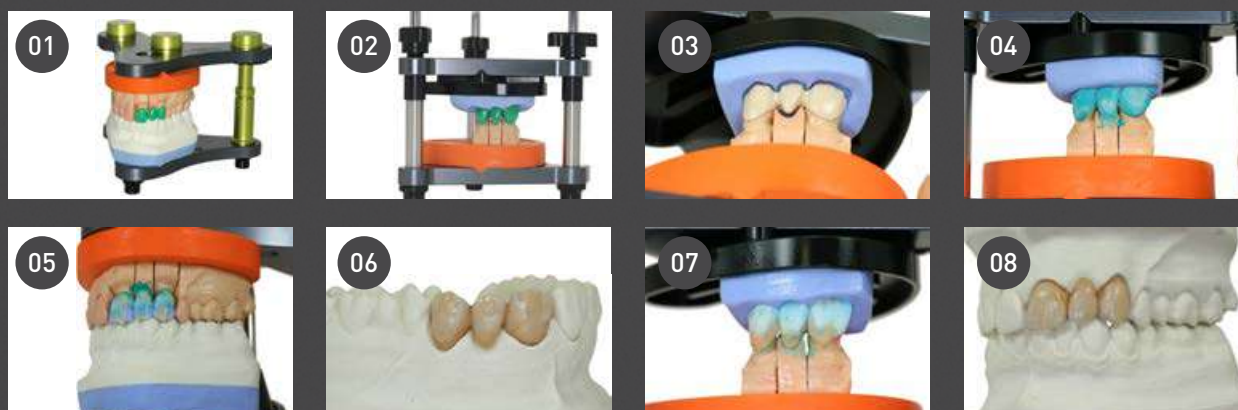- Universal Blue Silicone 70 Shore A+B

1. Occlusion check on the Articulator
2. Silicone template
3. Check of the spaces
4. Ceramic moulding
5. Dentine layering
6. First firing
7. Enamel layering
8. Final work after the second firing

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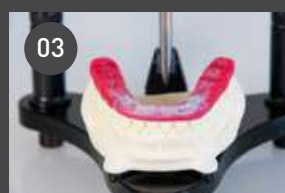
25min

~~60min~~

1. Occlusion check
2. Bite check on the Articulator
3. Bite modelling
4. First silicone template
5. Second silicone template
6. Flask opening
7. Isolation of the model
8. Preparation of the resin
9. Injection of the resin
10. Flask opening
11. Occlusion check on the Articulator
12. Final result after polishing

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Performed by: Dt. Luigi De Stefano

PRODUCTS USED:

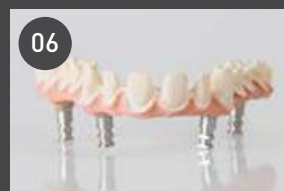
- Articulator
- Light Glass
- New Light Box

MATERIALS USED:

- Universal Blue Silicone 70 Shore A+B
- Traspasilicone 22 shore A+B
- Traspasilicone 50 shore A+B
- Comp Metal Primer
- Comp Metal Connector
- Comp Bonding
- Comp Flow
- Intensive Comp

3h

~~10h~~

1. Framework treatment and wax reproduction of the dental anatomy.
2. Setting of the wax-up on the BASE (B) with silicone and creation of injection canals. Traspa Silicone 22 shore pouring on COVER n°1 to obtain the template of the complete wax-up.
3. Positioning of the wax-up on COVER n°1, creation of injection canals and Traspa Silicone 50 shore pouring.
4. Wax-up set on the BASE (B) and cut-back.
5. Creation of injection canals for the enamel and closure of the flask with COVER n°2. Traspa Silicone 22 shore pouring.
6. Metal framework cleaned removing wax. Preparation of the metal framework using Metal Primer, Metal Connect and Comp Flow Opaquer.
7. Injection of Comp Flow Dentine on COVER n°2 and light-curing in the New Light Box.
8. Painting with Intensive Comp colours and light-curing in the New Light Box.
9. Injection of Comp Flow Incisal enamel on COVER n°1 and light-curing in the New Light Box.
10. Injection of Gengiva Flow composite on the BASE (B) and light-curing in the New Light Box.
11. Final light-curing in the New Light Box
12. Finishing and polishing.

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OUR VALUES

1

 We believe essentiality is the result of experience, research, collaboration and technology. It is a way of working with efficiency and efficacy.

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2

 Thanks to the high know-how of our team, we are able to design and produce innovative products that could facilitate your daily work. Everyone contributes with his own talent, diligence and expertise always considering the dental technician as the centre of each choice.

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3

 We listen to our customers needs to find together the best solution.

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4

 In the last 27 years the best materials and a constant innovation have always been our prerogatives to guarantee very high standards in every process.

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QUALITY AND CUSTOMER SERVICE

 Customer is at the core of all our choices: we grant high quality products and a constant service to support you in your daily operations avoiding wasting time and material. Our customer service:

- Pre-sale and post-sale assistance
- Constant innovation of our products, materials and accessories to improve our system
- Training courses and follow-up meetings with dental experts

Quality is made up of little details. That is why we use nickel-free anodised aluminium to assure you the unlimited life of our system.

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WORK

ENG

Fast Protec has an essential nature, since it synthesizes long and expensive traditional processing in a **fast, clean and easy** way of working. We invest in technology and regular research for constant innovation as aesthetics, ergonomics and functionality play decisive roles in reaching simplicity. As our approach always starts with dental technicians needs, we created a **complete system**, made up of flasks, articulators, verticulators, resins and silicones to perform all dental processing. This is Fast Protec: **precise protocols** that will guide you step by step to make your workflow faster and more organized.

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WHY TO CHOOSE
TRASFORMER-
FAST PROTEC

INDEX

<u>ENG</u>	<u>DEU</u>	<u>ESP</u>	
PRODUCTS	PRODUKTE	PRODUCTOS	8
Flasks	Muffel	Muflas	9
Articulator and Verticulators	Artikulator und Vertikulatoren	Articulador y Verticuladores	12
Implantology	Implantologie	Implantologia	13
Accessories	Zubehör	Accesorios	14
Polymerization Units	Polymerisierungsgeräte	Polimerizadoras	16
MATERIALS	MATERIALEN	MATERIALES	18
Silicones	Silikonen	Siliconas	19
Acrylics	Kunststoffe	Resinas	20
Isolating Agents	Isolierung	Aislantes	22
Composites	Kompositen	Composites	23
CAD/CAM Blanks	CAD/CAM Rohlinge	CAM/CAM Discos	27
DENTAL PROCESSING	BEARBEITUNGEN	ELABORACIONES	28
OUR VALUES	UNSERE WERTE	VALORES	42
PRODUCTS CODES	PRODUKTCODES	CÓDIGOS	43

PRODUCTS

PRODUKTE

PRODUCTOS

TRASFORMER

HAUTON

10

ALU BIG

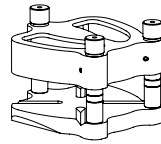
ENG - Alu Big is a flask made of aluminium with an ergonomic design.

By using silicone templates, it is possible to pour, inject or press the resin to obtain in a fast and precise way the most part of dental processing as removable dentures, relines, removable partial dentures, crown and bridges, surgical templates and implant dentures.

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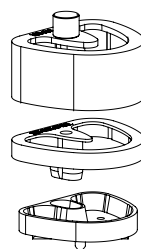
IMPLANT BOX

ENG - Implant Box is a flask with a minimal and compact design that allows you to cast the model in plaster directly on the base. Thanks to the integrated split cast it is possible to perfectly fit the base on Fast Protec Articulator, Verarte and All-In. It is provided with two tips: the metallic one is useful to perform crown and bridges and implant dentures whereas the clear one is perfect to work with light-curing composites.

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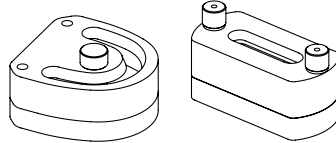
BIG AND SMALL GLASS

ENG - Big and Small Glass are transparent flasks with a POM base and two clear lids held by secure locking screws to ensure optimal results and no increases in the vertical dimension. The sizes of these flasks make them compatible with any polymerization chamber available on market and the double lids have been designed for the duplication of crown and bridge wax-up in light-curing composite.

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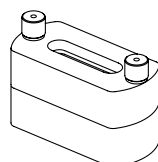
SMALL GLASS XL

ENG -Its compact shape, makes the Small Glass XL the ideal flask to realize works from 1 to 6 elements. Thanks to its additional lid, this flask allows technicians to perform implant dentures, joining the advantages of the Small Glass with those of the Light Glass, reducing any waste of Silicone and working on teeth and gingiva.

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
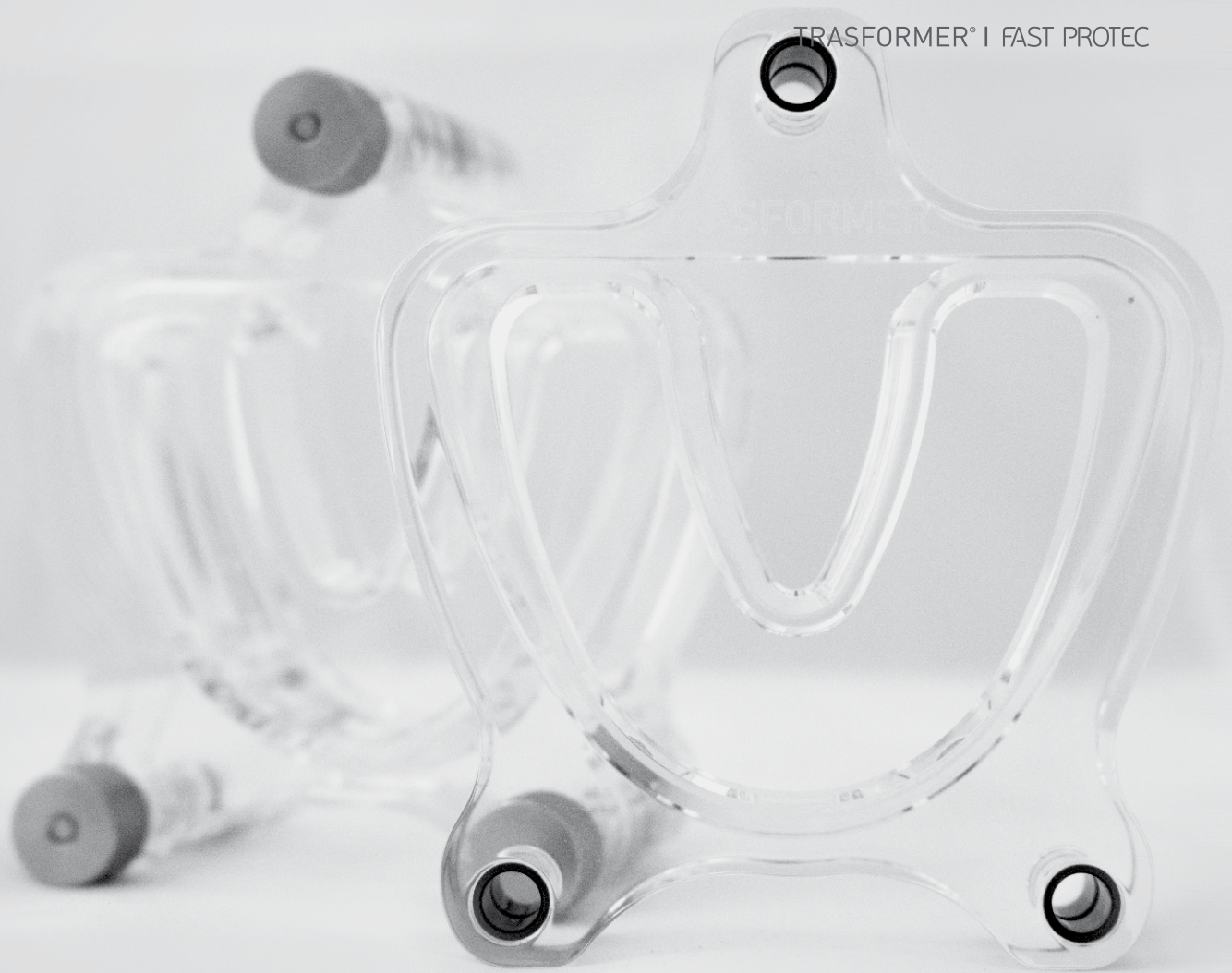
LIGHT GLASS

ENG - Wider than traditional transparent flasks and more stable with its three screws, Light Glass allows dental technicians to perform full implant dentures working simultaneously on composite teeth and gingiva thanks to the provided three lids and the removable three pins. Its completely clear structure guarantees a fast and total light-curing of the elements obtaining perfect and precise details.

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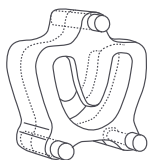ARTICULATOR

ENG - The unique articulator of this system is provided with magnetic condyles fixed at the standard 30 degrees to perfectly reproduce all mouth movements. Diagnosis and treatment planning are simplified thanks to the posterior incisal guide pin that gives total visibility at 360°.

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COD - 200.0009

VERARTE

ENG - Verticator and Articulator combined in one unit in a revolutionary design. The posterior rod pin guarantees a perfect and complete visibility of the prosthesis and simply by changing the upper support of the Articulator, it is possible to transform it in a verticator to perform ceramic moulding or plan implant dentures.

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COD - 200.0029

ALL IN

ENG - All-In is a combination of verticator and flask composed by one base with three rod pins and two interchangeable lids. The equilateral base enable the rotation of the model at 360°, allowing the dental technician to have the maximum visibility of his work. Essential item for the pre-visualization of the case and the casework planning as a verticator, it could be used for the injection or pouring technique as a flask.

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COD - 200.0030

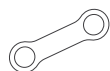
EASY BAR

ENG - Easy Bar is a kit of stainless steel preformed bars that facilitate the realization of a stable, precise and reinforced structure for all kind of fixed dental prosthetics in just 6 hours. Thanks to their different sizes (from 6 to 30 mm), these bars could be adapted by bending to any position of the implants and are compatible to every implant abutment on the market.

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COD - 300.00 - Stainless steel
400.00 - Titanium

EASY BENDER

ENG - This pair of pliers is born to complete the Easy Bar system for the construction of immediate loading frameworks. An original instrument to bend and adapt these preformed bars to any kind of implant position and type, easily obtaining the desired curve both longitudinally and horizontally. The comfortable grip of Easy Bender grants its flexibility and usability.

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» ACCESSORIES

DOUBLE BASE

ENG -Two base formers made of aluminium with rubber cuffs useful to cast the model in plaster. Thanks to the split cast designed on the base, the model becomes compatible with any product of the Fast Protec System fitting in a precise and clean way.

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COD - 200.1002

TEETH CLEAN

ENG -Basket with 14 individual compartments for steam cleaning teeth securing each denture tooth position.

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COD - 200.0102

ALU BIG CLEAR TOP

ENG -This clear top transforms Alu Big in a transparent flask to work with light-curing composites.

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COD - 200.0022.1

MODEL PLATE

ENG -Model Plate is a positioning device for setting edentulous models on the articulator.

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COD - 200.1007

CLEAN TEMP

ENG - Split cast base for temporaries crowns and bridges. Minimum waste of material thanks to its three undercuts.

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COD - 200.1005

BASE FOR PLASTER

ENG - Base for adapting any articulator on the market to the Fast Protec System.

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COD - 200.1003

UNIVERSAL CLEAN

ENG - Universal Clean is a little base that helps fitting the plaster model on the Articulator through the use of Universal Blue Silicone.

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COD - 200.1004

UNIVERSAL BASE SPLIT CAST

ENG - Little split cast base to adapt other bases available on market to the Fast Protec system.

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COD - 200.1001

DISPENSER EASY FLOW

ENG - Dispenser gun for syringes (3gr and 20gr) and cartridges.

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VICTORIA

NEW LIGHT BOX

ENG - The New Light Box is a compact and modern polymerization unit provided with last generation multi-die LEDs for light-curing composite resins. A low consumption machine that works on a vacuum pump to avoid the dispersion layer, the New Light Box is perfect to light-cure the Fast Protec wide range of composites at a wave frequency between 360 and 520 nm.

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COD - 200.0088

VICTORIA

ENG - Compact, easy and functional polymerizing machine. It is possible to cook cold and heat-curing resins with water input and to harden silicone templates with manual air loading through button up to 6 bar pressure. Victoria is provided with heat strips that heat water gradually until a maximum temperature of 120 degrees and an automatic counter that starts decreasing time once the pre-set temperature is reached maintaining it constant.

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A white
B blue

UNIVERSAL BLUE SILICONE

ENG - Kneadable silicone for the first templates. It guarantees an extremely precise detail reproduction without distortion even at high temperatures and an excellent long-term dimensional stability.

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A
B

PUTTY SILICONE

ENG - With ideal handling characteristics and times, Putty Silicone is an economic rigid bulk filler silicone.

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A
B

TRASPA SILICONE

ENG - Crystal-clear silicone perfect for processing with light-curing composites.

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A
B

REGULAR PRECISION

ENG - Auto-curing and precise silicone useful to reproduce temporaries caps or for temporaries restoration.

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ACRYLIC FOR DENTURES MICRO PEARL

ENG - Cold-curing resin for the injection, pouring or pressing techniques. Different colours available but the same stability and no shrinkage to meet all your needs: full and partial dentures, repairs and relining could be performed in an easy and fast way. Top aesthetic prosthesis with less residual monomer.

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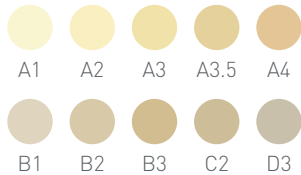
Incisal:

TEMP POWDER AND LIQUID

ENG - High-tech acrylic for long lasting crowns and bridges . Temp Powder is a cold-curing resin with a high precision and abrasion resistance that avoids plaque deposits.

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blue red white brown yellow

GENGICOLORS KIT

ENG - Specifically studied for Fast Protec acrylic micro pearled resins, Gengicolors is a kit composed by two pre-mixed light and dark cold-curing resins and five stain colours for the characterization of marginal and attached gingivas. Optimal to give dentures a natural and individualized look.

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CONTENT: 2 x powder 100 gr; liquid 100 ml; 8 x stains 10 gr

UNIPATT RESIN & UNIPATT GEL

ENG - Unipatt Resin is a self-curing castable resin for telescopes, super- and supra-structures using the brush technique.

Unipatt Gel is a light-curing gel-like resin for milling and precision attachments, intraoral connections and modelling. This material could be applied directly from the syringe, it burns without residue and could be polymerized with a standard light curing device (320-500 nm).

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COMP FLOW

COMP FLOW

ENG - Light-curing flowable composite resin easy to inject, finish and polish perfect for inlays, onlays, implant-supported super structures, laminate veneers and long term temporary restorations. Its fluorescence and opalescence guarantee individual and natural aesthetic results respecting functionality and biocompatibility. Using Fast Protec transparent flasks it is possible to realize few elements or a complete arch in a fast and absolutely precise way.

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GENGIVA FLOW

ENG - Light-curing gingiva composite ideal for gingiva restorations and reproductions, implant suprastructures, crowns and bridges. Thanks to Fast Protec wide range of colours, it is possible to characterize full or partial dentures with incredible final aesthetic results.

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CONTENT: 1 syringe 3g

COMP METAL PRIMER

ENG - Metal Primer is a perfect bonder between acrylics or composites and all types of metal.

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CONTENT: 5ml

COMP METAL CONNECTOR

ENG - Light-curing composite that improves the chemical bonding between primer and opaquers with a simple and secure adhesion.

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CONTENT: 1 syringe 3g

COMP BONDING

ENG - Conceived for the Fast Protec composite line, Bonding is a light-curing material for extensions and repairs of crown and bridge composites.

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CONTENT: 7ml

BOND OPAQUER PINK DARK

ENG - Light-curing paste opaquer useful to cover model casts with an integral adhesive component and perfect to cover Easy Bar frameworks before the injection of the resin.

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CONTENT: 1 syringe 3g

COMP LACQUER AND EVER LAQUER

ENG - Light-curing sealing lacquers for applying to the surfaces of denture acrylic, special tray material and composite crowns and bridges. Comp Lacquer is useful for temporary works whereas Ever Lacquer is a nifty product for permanent dentures.

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CONTENT: Comp Lacquer 10ml; Ever Lacquer 5 ml

COMP COVER

ENG - Masking gel that helps avoiding the dispersion layer when light-curing crown and bridge composites.

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CONTENT: 4g

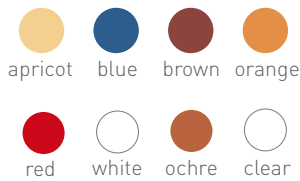
COMPOSITE WARMER

ENG - This warmer has been specifically studied for 3 and 20 grams composite syringes. An easier and faster injection is granted thanks to the two pre-set temperatures (39°C and 49°C) that reduce composite's viscosity and avoid bubbles.

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
INTENSIVE COMP KIT

ENG - Fast Protec offers a wide range of light-curing colours to characterize acrylic and composite teeth. Intensive Comp stains are easy to use because creamy and could be mixed to guarantee a natural and individual look according to the patient needs. The kit is provided with eight stain colours, a cleaner and a primer.

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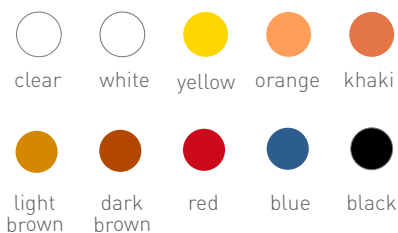CONTENT: 8 stains 10 g, 1 cleaner 50ml, 1 primer 3g.

STAIN FLOW KIT

ENG - Available in 1 gram syringes and 10 different colours, these stains will ease and fasten the individualization of composite crowns and bridges, full and partial dentures, acrylic teeth and permanent mouth restorations. They could be injected directly from the syringe or mixed on a palette and sliding brushed on inlays, onlays and veneers obtaining absolutely natural shades.

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MASTER POLYURETHANE

ENG - Polyurethane resin ideal for fabricating precise and resistant models as well as crowns and bridges. Flowable and easy to trim, Master Polyurethane has a dimensional stability that guarantees precise details of all works with a shrinkage percentage close to zero.

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» ISOLATING AGENTS | ISOLIERUNG | AISLANTES

**FAST ALGINATE**

ENG - Separating agent, based on alginate, useful to isolate the plaster model before injecting or pouring the resin.

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TITANIUM BLANKS

ENG - Titanium alloy discs Ti6Al4V ELI Grade 23 (with 6% Aluminum, 4% Vanadium and Extra Low Interstitials) are ideal for creating metal structures for dental prostheses using CAD CAM milling systems.

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CR-CO BLANKS

ENG - The Cobalt-Chrome blanks are suitable for the construction of prosthetic, telescopic and anatomical structures using the milling technique. The excellent mechanical properties and the low oxide formation make them free from toxic and corrosion and heat resistant elements.

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COMPOSITE BLANKS

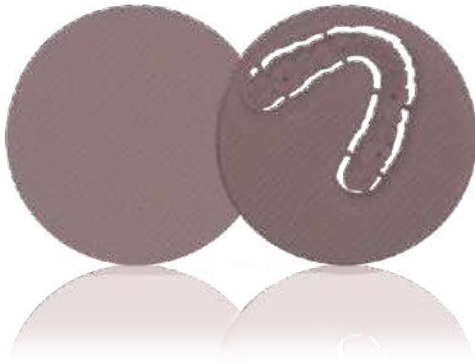
ENG - CAD/CAM Composite Blank is the result of an innovative combination of materials offering both the advantages of high-performance acrylics (PMMA) and flexible zirconium dioxide. A further step to perfection: high elasticity that reproduces the natural tooth and high abrasion resistance, that means a flexible framework that distributes pressure evenly and reduces the strain on jaw bones and joints.

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COD - CM98.20.A1 | CM98.20.A2 | CM98.20.A3

GLASS FIBER BLANKS


ENG - The Glass Fiber overlapping layers are pressed with a special machine with over 20 tons of pressure, in this way it is possible to compress a large amount of material in a decidedly space content. Biocompatible, fiber glass is now used with great success for many applications in dentistry.

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COD - GF.XX - H. disponibili: 10, 15, 20 mm

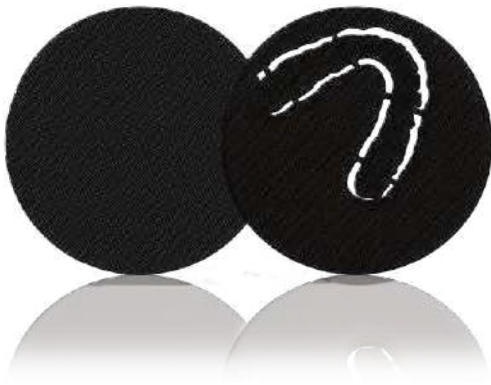CARBON FIBER BLANKS

ENG - CAD/CAM Carbon Fiber blank is composed by over 100 layers of superimposed fiber pressed and oriented in a particular way in order to obtain the maximum mechanical performance. With only 1.7 gr/cm of specific weight this is the lighter material that we can find for dental prostheses.

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45 MIN

ENG

1. Wax-up
2. Occlusion check on the Articulator
3. First silicone template on the Alu Big
4. Second template
5. Teeth degreasing
6. Repositioning of the teeth on the silicone template
6. Isolation of the model
7. Closing of the flask
8. Resin pouring
9. Opening of the flask
10. Polishing
11. Final occlusion check

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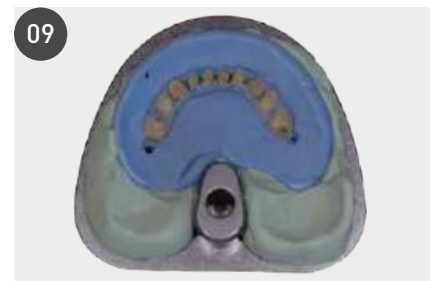
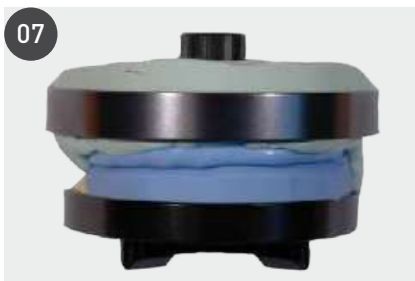
6 H

ENG

1. Implant denture model
2. Control of the bar position (previously fixed in mouth)
3. Model set on Articulator
4. Easy Bar structure welded
5. Easy Bar structure cemented on the abutments
6. Wax-up
7. Silicone templates
8. Opacified structure
9. Teeth insertion
10. Injection of the resin
11. Opening of the muffle
12. Final result after polishing

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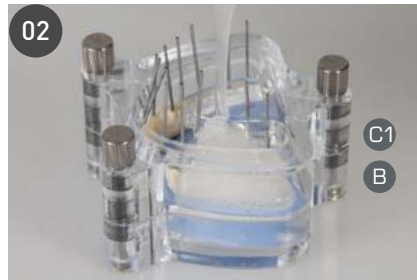
6 H

ENG

1. Framework treatment and wax-up
2. Setting of the wax-up on the BASE (B) with silicone and creation of injection canals. Traspas Silicone 22 shore pouring
3. Positioning of the wax-up on COVER n°1, creation of injection canals and Traspas Silicone 50 shore pouring.
4. Wax-up set on the BASE (B) and cut-back.
5. Creation of injection canals for the enamel and closure of the flask with COVER n°2.
6. Preparation of the metal framework using Metal Primer, Metal Connect and Comp Flow Opaquer.
7. Injection of Comp Flow Dentine on COVER n°2 and light-curing
8. Light-curing in the New Light Box.
9. Painting with Intensive Comp colours .
10. Injection of Comp Flow Incisal enamel on COVER n°1 and light-curing
11. Injection of Gengiva Flow composite and light-curing.
12. Finishing and polishing.

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OUR VALUES

1

ENG - We believe essentiality is the result of experience, research, collaboration and technology. It is a way of working with efficiency and efficacy.

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2

ENG - Thanks to the high know-how of our team, we are able to design and produce innovative products that could facilitate your daily work. Everyone contributes with his own talent, diligence and expertise always considering the dental technician as the centre of each choice.

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3

ENG - We listen to our customers needs to find together the best solution.

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4

ENG - In the last 25 years the best materials and a constant innovation have always been our prerogatives to guarantee very high standards in every process.

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Code	Description	Content
1912028	Intensive Comp - Apricot	3g
1912029	Intensive Comp - Blue	3g
1912037	Intensive Comp - Brown	3g
1912031	Intensive Comp - Orange	3g
1912032	Intensive Comp - Red	3g
1912033	Intensive Comp - White	3g
1912034	Intensive Comp - Ochre	3g
1912035	Intensive Comp - Clear	3g
1912036	Intensive Comp Kit	1
1912050	Stain Flow - Clear	1g
1912051	Stain Flow - White	1g
1912052	Stain Flow - Yellow	1g
1912053	Stain Flow - Orange	1g
1912054	Stain Flow - Red	1g
1912055	Stain Flow - Khaki	1g
1912056	Stain Flow - Blue	1g
1912057	Stain Flow - Light Brown	1g
1912958	Stain Flow - Dark Brown	1g
1912059	Stain Flow - Black	1g
200.0089	Composite Warmer	1
CM98.20.A2	CAD/CAM Composite Blank 98x20xmm A2	1
CM98.20.A3	CAD/CAM Composite Blank 98x20xmm A3	1
T98.XX	CAD/CAM Titanium Blank 98mm diameter - XX = Height (8, 10, 12, 13.5, 15, 16, 18, 20 mm)	1
CR98.XX	CAD/CAM Cobalt Chrome Blank 98mm diameter - XX = Height (10, 12, 13.5, 15, 18, 20 mm)	1
CB98.XX	CAD/CAM Carbon Fiber Blank 98mm diameter - XX = Height (10,15, 20 mm)	1
GF98.XX	CAD/CAM Glass Fiber Blank 98mm diameter - XX = Height (10, 15, 20 mm)	1

TIP CHARTS

COMPOSITE COMP FLOW

Vita® Colour	Comp Flow Opaquer	Comp Flow Dentina	Comp Flow Incisal
A1	Light	A1/B1	S58
A2	Light	A2	S58
A3	Light	A3	S59
A3.5	Medium	A3.5	S59
A4	Medium	A4	S60
B1	Light	A1/B1	S57
B3	Light	B3/B4	S59
B4	Medium	B3/B4	S59
C2	Medium	C2/C3	S59
C3	Dark	C2/C3	S59
D2	Light	D2/D3	S60
D3	Dark	D2/D3	S59

RESIN TRAS TEMP

Vita® Colour	Tras Temp
A1	SE3
A2	SE2
A3	SE2
A3.5	SE1-2
A4	SE1-2
B1	SE1
B2	SE1-2
B3	SE1
C2	SE1
D3	SE2

Thanks to:

Vincenzo De Feo and Umberto Scilipoti
for the provided images of dental processing.

We would like to thank Vincenzo Cuiuli,
inventor of the system, for his unique contribution.

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are trademarks of PPM srl